


# HOW TO WEAR A N95 RESPIRATOR




WASH YOUR HANDS

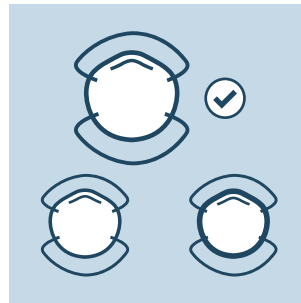

CHOOSE A MASK SIZE  
AND MODEL  
THAT FITS YOU WELL



HOLD THE MASK IN YOUR  
HAND AND PLACE IT ON  
NOSE, MOUTH AND CHIN

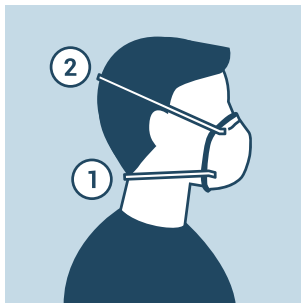

POSITION THE BOTTOM  
STRAP UNDER YOUR EARS  
AND THE TOP STRAP AT THE  
BACK OF YOUR HEAD



PRESS THE METALLIC WIRE  
TO FIT THE SHAPE  
OF YOUR NOSE AND FACE



PERFORM A FIT CHECK:  
PLACE HANDS OVER THE  
RESPIRATOR AND EXHALE,  
ADJUST  
IF THERE ARE LEAKS